

TECHNICAL SPECIFICATIONS

COMFORT VALUE Range
Shower Chair
(CVSC/ATT &
CVSC/SP)

ISSUE DATE: 11-10-19
VERSION: TS/CVSC-01
STATUS: APPROVED
AUTHORISED BY: Technical Director



K.YOUNG



CONTENTS

1. Application
2. Features Overview
3. Product Dimensions
4. Warranty Information
5. Product Parts List
6. Other supporting documentation

1. APPLICATION

The COMFORT VALUE Shower chair is designed to be used as a self-propelling or attendant operated. All the parts being interchangeable to facilitate simple change of operation should this be required






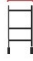





2. FEATURES OVERVIEW

APPLIES TO BOTH CVSC/ATT & CVSC/SP	
Height Adjustable Frame	3" adjustment (76mm)
Wheel configuration	4 x 5" (125mm) dia. braked castors OR 2 x 22" (560mm) Self propelling rear wheels with hand rim and 2 x 5" (125mm) dia. braked castors
Seat type	Removable / Reversible wipe clean, vinyl upholstered comfort gap seat
Back Rest	Tension adjustable wipe clean vinyl
Foot Rests	Height adjustable/swing away/ folding wheelchair style
Head Rest	Fully removable wipe clean vinyl
Arm Rests	Rise & Fall movement, self-supporting fully padded with hand grips
Frame Material	25mm dia. Stainless steel tubular frame, white powder coated
Fixtures & Fittings	All either fully galvanised / stainless steel materials
Colour	Black Seat, Grey backrest, Black footrest plates
Manufacture	Fabricated & Assembled in the UK by Care & Independence
Warranty	12 months
Wash Care	All parts can be wiped clean using mild soap solution / cleaning agent. Do not use bleach. Do not immerse. Wipe dry after washing. Suitable for decontamination and up to 80°C wash temps.

ACCESSORIES	
Lap strap	Up to 44" waist as standard (longer by quote). Black safety-belt material with side squeeze plastic clip.
Commode Pan/Lid/ Rack	3.5L capacity, oval shaped, unisex design vessel complete with anti-spill lid and fittings.
Foot Strap	Footplate mounted webbing straps with hook & loop adjustment.
Thoracic Support	Wheelchair style harness w/ waist belt, shoulder straps, groin strap to assist in maintaining an upright seated position.
Lateral Head Support	Removable wedge to fit headrest
Extended Warranty	Available on request by quote

3. PRODUCT DIMENSIONS

SELF-PROPELLED (CVSC/SP)												
Seat Size	17"		19"		21"		Measurements applicable for all seat sizing					
Product code	CVSC/SP-17		CVSC/SP-19		CVSC/SP-21		Seat Height	Overall Height	Seat Depth	Overall Depth c/w Footrests	Seat to footrests	SWL
	Seat Width	Overall Width	Seat Width	Overall Width	Seat Width	Overall Width	ADJ	1000mm	406mm	1000mm	ADJ	150kg
	432mm	650mm	483mm	700mm	533mm	750mm						

ATTENDANT (CVSC/ATT)												
Seat Size	17"		19"		21"		Measurements applicable for all seat sizing					
Product code	CVSC/ATT-17		CVSC/ATT-19		CVSC/ATT-21		Seat Height	Overall Height	Seat Depth	Overall Depth c/w Footrests	Seat to footrests	SWL
	Seat Width 432mm 	Overall Width 550mm 	Seat Width 483mm 	Overall Width 600mm 	Seat Width 533mm 	Overall Width 650mm 	ADJ 	1000mm 	406mm 	900mm 	ADJ 	17" 180kg 19" 180kg 21" 200kg

4. WARRANTY INFORMATION

This product is offered with a standard 12 months warranty from the date of purchase, subject to the following general terms and conditions:

- i. Should problems be experienced with any workmanship or materials within the first 12 months please contact your point of purchase or care and Independence.
- ii. Please quote the serial number / Batch of the product (where applicable), which will be found on the front cover on this user guide or on the product itself.
- iii. The product will be repaired free of charge provided that it has been used in accordance with the guidelines and recommendations set out in the user guide, and maintained by Care and independence or a competent person
- iv. The warranty does not cover replacements; adjustments, repairs arising from a consequence of normal wear & tear, wilful damage, accidental damage, misuse, neglect or any cause, which is beyond the manufacturers control.
- v. The warranty does not cover modifications or repairs made to the product by unauthorised persons or the use of unauthorised parts or any failure or malfunction due to incorrect and/or poor workmanship on the part of any person, other than Care and independence or a competent person
- vi. This warranty does not affect your statutory consumer rights.

5. PRODUCT PARTS LIST (CVSC/ATT & CVSC/SP)

ACCESSORIES (CVSC/ATT & CVSC/SP)	
PART NUMBER	TITLE
PCOMPAN	Commode Pan & Rack
LAPSTRAP/BK	Black Polyester Webbing Lap strap with Buckle
ACLATSUP48B	Thoracic Support - Black
FOOTSTRAP	Foot Straps

SPARE PARTS (CVSC/ATT & CVSC/SP)	
PART NUMBER	TITLE
CVSC/001	Backrest 17"
CVSC/002	Backrest 19"
CVSC/003	Backrest 21"
CVGF/17	Gap Front Seat - 17"
CVGF/19	Gap Front Seat - 19"
CVGF/21	Gap Front Seat - 21"
CV/001	Castor - Single Unit
C22SPWHEEL	22" Self-Propelling Wheel - Single Unit
BRAKE/LHS	Wheel Brake - LHS
BRAKE/RHS	Wheel Brake - RHS
CV/002	Footrest -Pair
PCOMRACK	Commode Pan Rack - Single Unit
PASCLIP	E Clip - single
R1BHG	Hand Grip
RBKARMPAD	Rubber Arm Pad

6. OTHER SUPPORTING DOCUMENTATION

- User Guide - available as a download from www.careandindependence.com
- Pricing - available from the relevant Care & Independence Area Manager